



# CPAP BATTERY

*The freedom to power your PAP device when and where you need it!*

## PRODUCT FEATURES

- Provides 12V direct current to operate most makes and models of PAP devices. \*Additional power component may be required.
- Use in a single battery configuration or pigtail two batteries together for even more power.
- Four stage LED power gauge indicates remaining Battery power level.
- Meets FAA requirements for lithium-ion batteries taken in carry-on baggage.
- Use as an emergency power backup during power outages.
- Power other small electronic devices such as cell phones, iPads and laptops.
- Options for charging the Battery via the cigarette lighter/power socket of a vehicle or by using the power of the sun.



**MODEL:** C-100

**DIMENSIONS:**

7.5" x 4.25" x 0.75" (L x W x H)

**WEIGHT:** 1.6 lbs

**CHARGE TIME:**

4-5 hours (per battery)

## COMPATIBILITY & AVERAGE RUN TIMES

Multiple factors will affect how long the CPAP Battery will power a PAP device including, but not limited to, device type (CPAP, BiLevel, AutoPAP, etc.), pressure setting, altitude, humidifier usage, and if additional power components are required in conjunction with the CPAP Battery. The use of heated humidification or heated tubing will significantly reduce run times and could prevent the CPAP Battery from being able to power the PAP device. BiLevel devices will generally have slightly lower run times as the pressure setting of these devices is not constant and changes throughout the night which requires more power to operate the device. Run times listed below are based on CPAP devices with no heated humidification or heated tubing.

### RESPIRONICS

#### DreamStation Series

{requires 12V power adapter: BPS C-12V-DS}

Single Battery: 1-2 nights

Dual Battery Configuration: 2-4 nights

#### M Series & System One Series

{no additional power components required}

Single Battery: 1-3 nights

Dual Battery Configuration: 3-5 nights

### RESMED

#### AirSense/AirCurve 10 Series

{requires 24V power converter: BPS C-24V-RES}

Single Battery: 1-2 nights

Dual Battery Configuration: 2-4 nights

#### S9 Series

{requires 24V power converter: BPS C-24V-RES}

Single Battery: 1-2 nights

Dual Battery Configuration: 2-4 nights

#### S8 Series

{requires 150W power inverter: BPS 150 INVERTER}

Single Battery: 1 night

Dual Battery Configuration: 2 nights

### DEVILBISS

#### IntelliPAP Series

{no additional power components required}

Single Battery: 1-3 nights

Dual Battery Configuration: 3-5 nights

### FISHER & PAYKEL | 3B MEDICAL | APEX MEDICAL

#### All models from these manufacturers

{requires 150W power inverter: BPS 150 INVERTER}

Single Battery: 1 night

Dual Battery Configuration: 2 nights

\*All power converters, adapters and inverters are sold separately and are not included in the standard C-100 CPAP Battery Kit.