

# Respironics DreamStation Series: Single Battery Connection


The DC input port on Respironics DreamStation series devices require a proprietary signal in order to recognize a valid power source; therefore, the use of a 12V power adapter is required in conjunction with the Freedom™ CPAP Battery. Our 12V power adapter was designed specifically for use with Respironics DreamStation series devices and the Freedom™ CPAP Battery.

## REQUIRED COMPONENTS

- Freedom™ CPAP Battery Kit
- DreamStation 12V Power Adapter

## CONNECTION INSTRUCTIONS

- Plug the 12V power adapter into the DC OUT jack of the battery.
- Plug the 12V power adapter into the DC input port of the PAP device.
- Turn the PAP device on and it should begin functioning.
- The digital display screen will show the charge level of the battery. You may press the charge level indicator button to illuminate this screen. It will stay illuminated for a few seconds at a time.
- When not in use disconnect all devices from the battery thus allowing the battery to completely shut down.

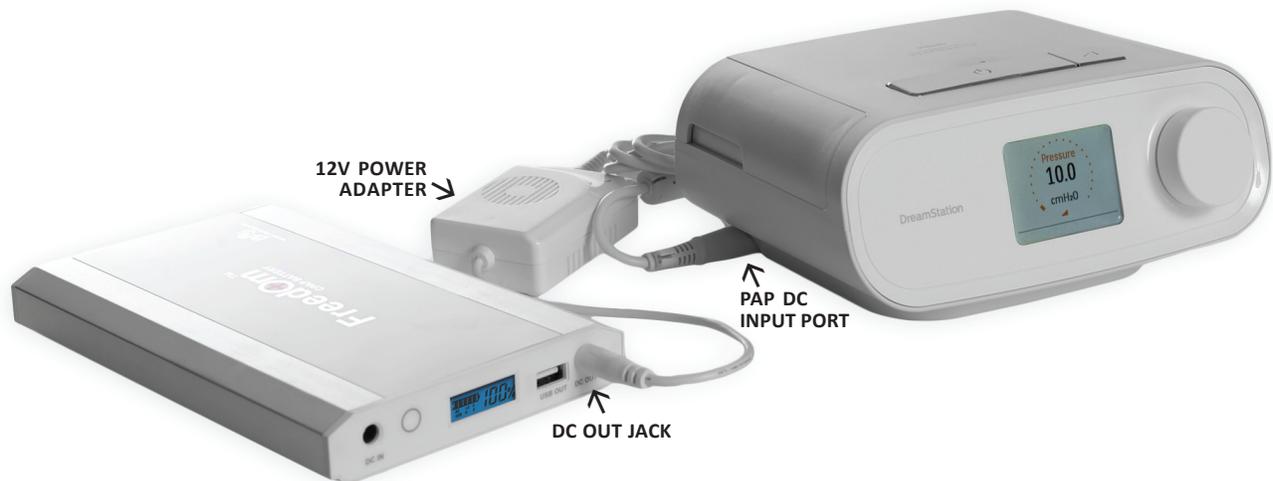

## OPTIONAL FREEDOM™ CPAP BATTERY CHARGING ACCESSORIES

- Auto Charge DC Cable {BPS DC.CABLE}: Used to charge the battery in a vehicle (vehicle must be running).
- 150W Sine Wave Power Inverter {BPS 150 INVERTER}: Used to charge the battery in a vehicle (vehicle does not need to be running) and allows the battery to power/charge other small electronic devices.
- SunPower 40W Solar Panel {BPS SP40}: Used to charge the battery using the power of the sun.

# Respironics DreamStation Series: Dual Battery Connection


The DC input port on Respironics DreamStation series devices require a proprietary signal in order to recognize a valid power source; therefore, the use of a 12V power adapter is required in conjunction with the Freedom™ CPAP Battery. Our 12V power adapter was designed specifically for use with Respironics DreamStation series devices and the Freedom™ CPAP Battery.

## REQUIRED COMPONENTS

- Freedom™ CPAP Battery Kit
- Freedom™ CPAP Battery (additional battery only)
- DreamStation 12V Power Adapter

## CONNECTION INSTRUCTIONS

- Plug the 12V power adapter into the DC power pigtail cord.
- Plug the DC power pigtail cord into the DC OUT jacks of both batteries.
- Plug the 12V power adapter into the DC input port of the PAP device.
- Turn the PAP device on and it should begin functioning.
- The digital display screen will show the charge level of the battery. You may press the charge level indicator button to illuminate this screen. It will stay illuminated for a few seconds at a time.
- When not in use disconnect all devices from the battery thus allowing the battery to completely shut down.

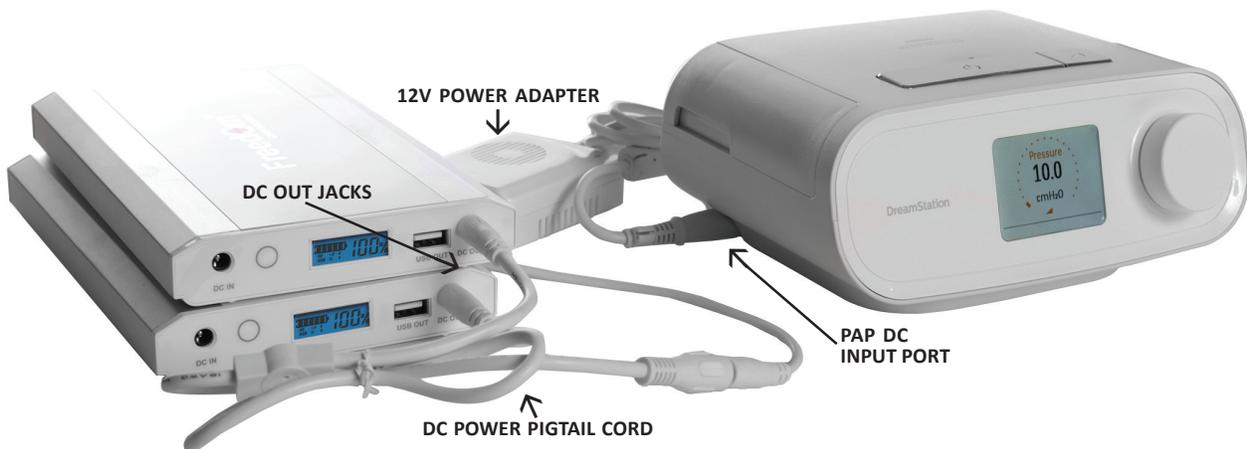

## OPTIONAL FREEDOM™ CPAP BATTERY CHARGING ACCESSORIES

- Auto Charge DC Cable {BPS DC.CABLE}: Used to charge the battery in a vehicle (vehicle must be running).
- 150W Sine Wave Power Inverter {BPS 150 INVERTER}: Used to charge the battery in a vehicle (vehicle does not need to be running) and allows the battery to power/charge other small electronic devices.
- SunPower 40W Solar Panel {BPS SP40}: Used to charge the battery using the power of the sun.