


Power Backup: Respiroics DreamStation Series

REQUIRED COMPONENTS

- Freedom™ CPAP Battery Kit
- DreamStation 12V Power Adapter

SINGLE BATTERY CONNECTION

- Plug the battery AC power supply into the DC IN jack of the battery.
- Plug the AC power supply into a wall outlet.
- Plug the 12V power adapter into the DC OUT jack of the battery.
- Plug the 12V power adapter into the DC input port of the PAP device.
- The digital display screen will show the charge level of the battery. Press the charge level indicator button to illuminate the screen.

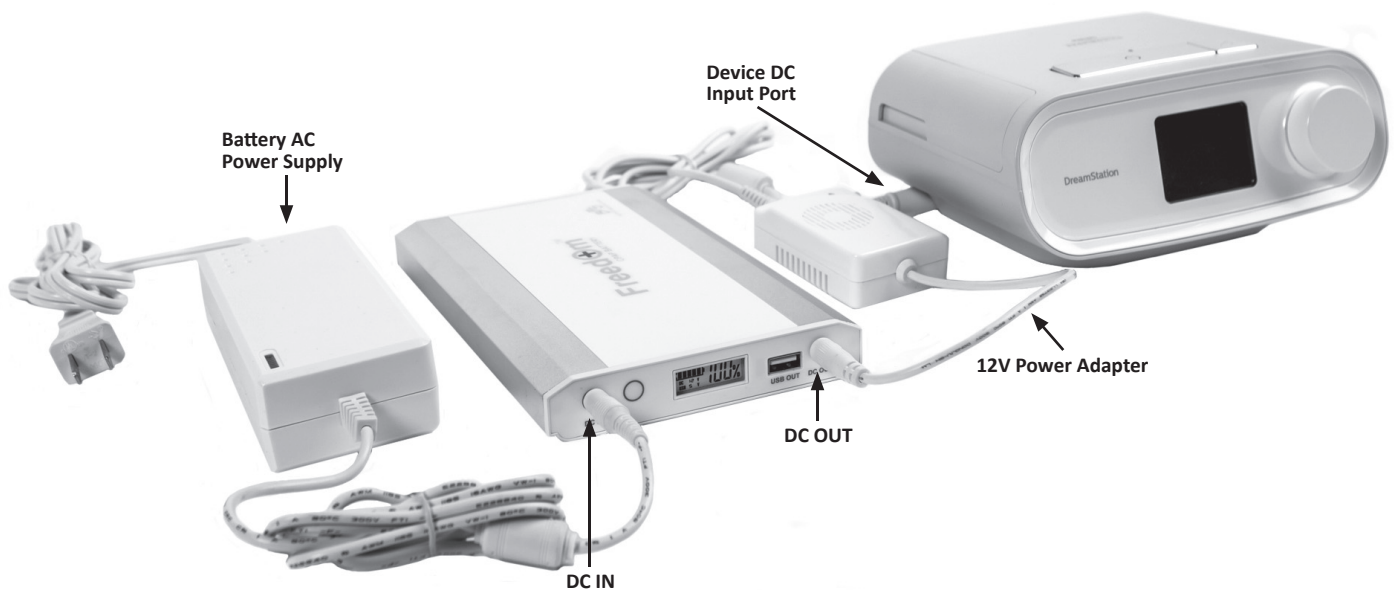

Power Backup: Respironics DreamStation Series

DUAL BATTERY CONNECTION

*Dual battery connection will require an additional Freedom™ battery and an additional DC power pigtail cord (item # BPS PIGTAIL.FD).

- Plug the battery AC power supply into the first DC power pigtail cord.
- Plug the first DC power pigtail cord into the DC IN jacks of the batteries.
- Plug the AC power supply into a wall outlet.
- Plug the second DC power pigtail cord into the DC OUT jacks of the batteries.
- Plug the 12V power adapter into the second DC power pigtail cord.
- Plug the 12V power adapter into the DC input port of the PAP device.
- The digital display screen will show the charge level of the battery. Press the charge level indicator button to illuminate the screen.

