

# Power Backup: Other PAP Devices

---

## REQUIRED COMPONENTS

- Freedom™ CPAP Battery Kit
- 150W Sine Wave Power Inverter (power inverter)

## SINGLE BATTERY CONNECTION

- Plug the battery AC power supply into the DC IN jack of the battery.
- Plug the AC power supply into a wall outlet.
- Plug the DC input cord into the DC OUT jack of the battery.
- Plug the power inverter into the DC input cord.
- Plug the AC power cord of the PAP device into the power inverter.
- Plug the AC power cord into the PAP device.
- The digital display screen will show the charge level of the battery. Press the charge level indicator button to illuminate the screen.

\*Image example shows a 3B Medical Luna device.



# Power Backup: Other PAP Devices

## DUAL BATTERY CONNECTION

\*Dual battery connection will require an additional Freedom™ battery and an additional DC power pigtail cord (item # BPS PIGTAIL.FD).

- Plug the battery AC power supply into the first DC power pigtail cord.
- Plug the first DC power pigtail cord into the DC IN jacks of the batteries.
- Plug the AC power supply into a wall outlet.
- Plug the DC input cord into the second DC power pigtail cord.
- Plug the second DC power pigtail cord into the DC OUT jacks of the batteries.
- Plug the power inverter into the DC input cord.
- Plug the AC power cord of the PAP device into the power inverter.
- Plug the AC power cord into the PAP device.
- The digital display screen will show the charge level of the battery. Press the charge level indicator button to illuminate the screen.

\*Image example shows a 3B Medical Luna device.

