

Power Backup: Respiroics System One & M Series | DeVilbiss IntelliPAP Series


REQUIRED COMPONENTS

- Freedom™ CPAP Battery Kit

SINGLE BATTERY CONNECTION

- Plug the battery AC power supply into the DC IN jack of the battery.
- Plug the AC power supply into a wall outlet.
- Plug the DC power pigtail cord into the DC OUT jack of the battery.
- Insert the appropriate connector tip into the DC power pigtail cord.
 - System One 60 Series: Tip with black top labeled “W”
 - System One 50 & M Series: Tip with yellow top labeled “N”
 - IntelliPAP Series: Tip with blue top labeled “J”
- Plug the DC power pigtail cord with appropriate connector tip attached into the DC input port of the PAP device.
- The digital display screen will show the charge level of the battery. Press the charge level indicator button to illuminate the screen.

*Image shows connection to a Respiroics System One PAP device.

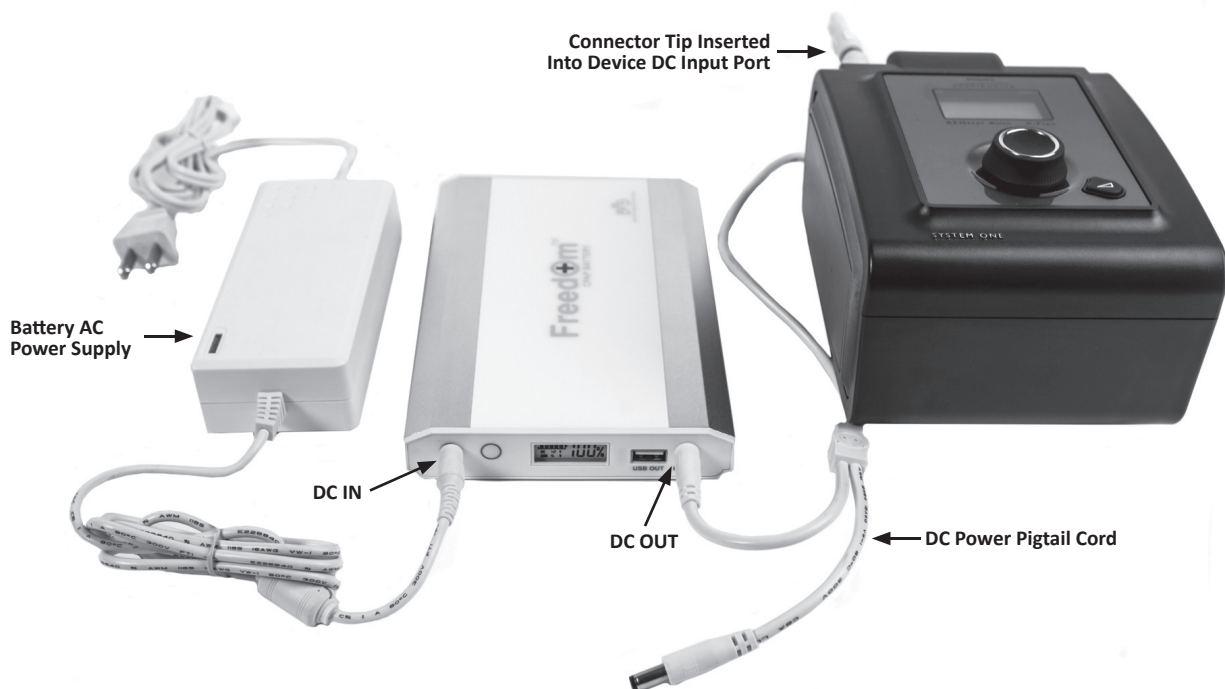

*Note: DeVilbiss IntelliPAP 2 series devices will connect to the Freedom™ battery via the 150W sine wave power inverter


Power Backup: Respiroics System One & M Series | DeVilbiss IntelliPAP Series

DUAL BATTERY CONNECTION

*Dual battery connection will require an additional Freedom™ battery and an additional DC power pigtail cord (item # BPS PIGTAIL.FD).

- Plug the battery AC power supply into the first DC power pigtail cord.
- Plug the first DC power pigtail cord into the DC IN jacks of the batteries.
- Plug the AC power supply into a wall outlet.
- Plug the second DC power pigtail cord into the DC OUT jacks of the batteries.
- Insert the appropriate connector tip into the second DC power pigtail cord.
 - System One 60 Series: Tip with black top labeled “W”
 - System One 50 & M Series: Tip with yellow top labeled “N”
 - IntelliPAP Series: Tip with blue top labeled “J”
- Plug the second DC power pigtail cord with correct connector tip attached into the DC input port of the PAP device.
- The digital display screen will show the charge level of the battery. Press the charge level indicator button to illuminate the screen.

*Image shows connection to a Respiroics System One PAP device.

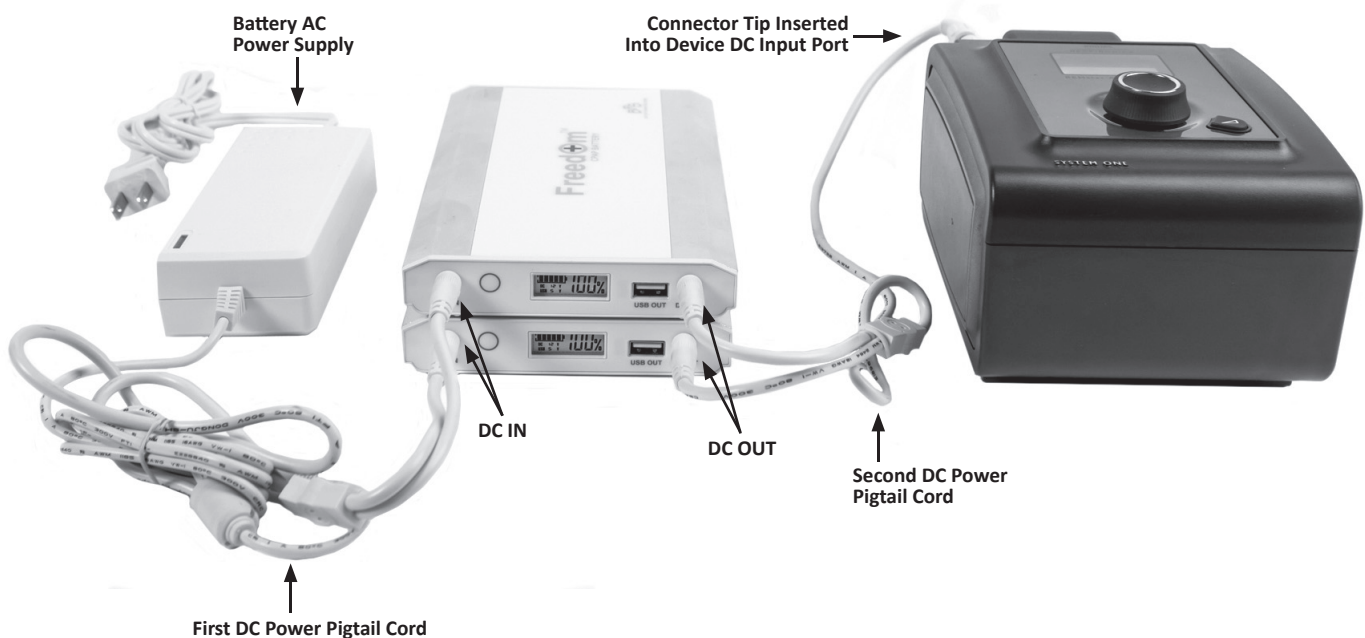

*Note: DeVilbiss IntelliPAP 2 series devices will connect to the Freedom™ battery via the 150W sine wave power inverter